



Setting the time and date


1. From the HOME screen, press ACT. Press the  button to select Utilities. Press ACT.




2. Press the  button to select Time/Date. Press ACT.



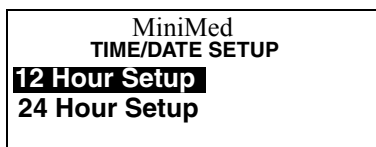


3. Select your preferred time format. Press ACT.

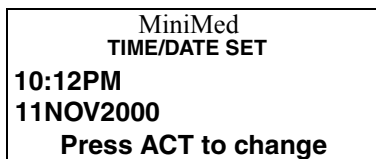

4. Press ACT to program the time and date.



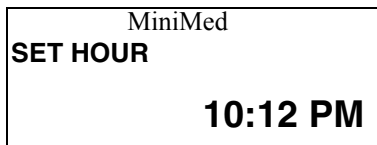


5. The hour will flash. Press the  and  buttons to select the hour. Press ACT to set the hour.




Note:



When pressing the  and  buttons the AM/PM icon will change. Make sure that the desired icon is displayed.

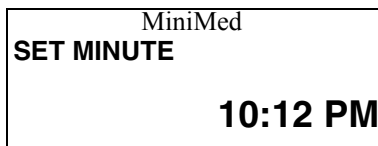
6. The minutes will flash. Press the  and  buttons to select the minutes. Press ACT.



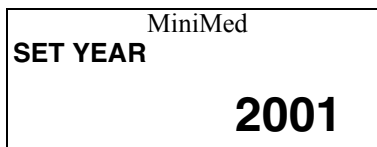

7. The year will flash. Press the  and  buttons to select the current year. Press ACT.

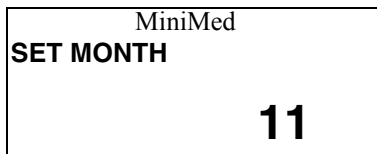

8. The month will flash. Press the  and  buttons to select the month. Press ACT.



9. The day will flash. Use the  and  buttons to select the current day. Press ACT.



10. The time and date will be shown.

