


SET CHANGE REMINDER IN THE MINIMED™ 640G INSULIN PUMP




The Set Change reminder in the MiniMed 640G helps you remember to change your infusion set. After you turn on this reminder, it automatically tracks the time between infusion set changes and reminds you to change your infusion set.

This is how to turn on or off, or change the Set Change Reminder

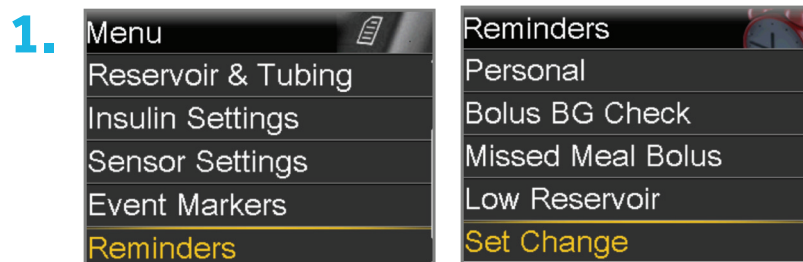


Go to the Set Change screen.

Menu > Reminders > Set Change

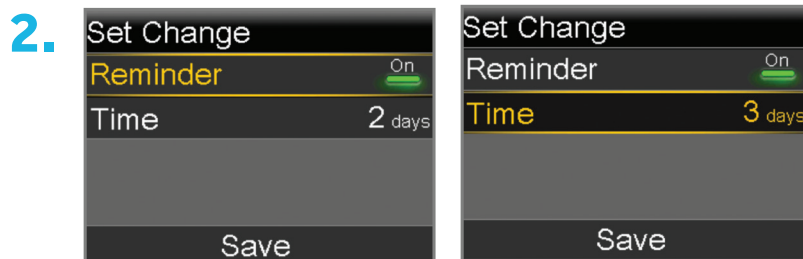

Select Reminder to turn the reminder on or off. If you turn on the reminder, select Time and choose two or three days for the reminder.

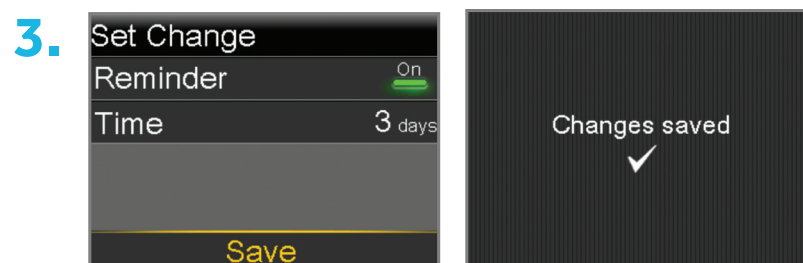

Select Save.

Safety Information: MiniMed™ 640G Insulin Pump is indicated for the continuous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin.

For a listing of indications, contraindications, precautions, warnings, and potential adverse events, please refer to the Instructions for Use.

The content and all information provided here is for your informational use only and is not intended to be a substitute for professional medical advice, diagnosis or treatment in any manner.

UC201607843EE © 2016 Medtronic.
MiniMed 640G is a trademark of Medtronic, Inc. All rights reserved. Printed in Europe.

Medtronic
Further, Together